



① PRE-SOAK

- Pre-soak instruments and surgical kit in an enzymatic detergent immediately after surgical procedure for 20 minutes.
- Reamers and other small parts should soak separately from the surgical kit.

② CLEAN MANUALLY

- Surgical instruments and kit should then be cleaned manually with a soft brush to remove all debris.
- To thoroughly clean the Inserter/Retriever, we recommend separating the two components

③ CLEAN ULTRASONICALLY

- Clean instruments and kit in an ultrasonic machine filled with an anti-corrosive enzymatic detergent on the longest cycle.

④ REASSEMBLE KIT

- Place all instruments and small parts back into surgical kit.
- Place autoclave indicator strip in kit.
- Close kit.

⑤ WRAP KIT

- Double wrap instrument kit with autoclave paper wrap and seal with autoclave indicator tape

⑥ STERILIZE

- Sterilize in an autoclave for 30 minutes at 273° F (134°C).
- Run autoclave dry cycle for a minimum of 20 minutes.



DO NOT COLD STERILIZE SURGICAL INSTRUMENTS OR SOAK IN HYDROGEN PEROXIDE, BLEACH, OR ALCOHOL.